

Dear Customer

Thank you for purchasing the Half Step.

Entrances where the step is too high can be tricky, especially if you suffer from arthritis, rheumatism or sports injuries. This Half Step meets the problem half way (quite literally) by providing an intermediate level between the threshold and the ground. The large slip-resistant platform is specially designed to allow you to lift your feet and remain steady while stepping up. Strong and sturdy to support up to 200kgs (31.4 stone), it's heavy-duty, yet portable, so easy to move if desired. Made from rust free, weather-resistant material, it can be used indoors or outside your front or back door, thanks to 'invisible' drainage holes. Measures L49xW39xH10cm.

IMPORTANT

- Always use extreme caution when walking up or down the step.
- Use extra caution if step is wet.

BEFORE USE

- 1) Check the item as soon as it arrives.
- 2) Do not use the item if it has been damaged during shipping.

INSTRUCTIONS FOR USE

- 1) Place the Half Step directly in front of your doorway.
- 2) Ensure that the Half Step is placed on a flat surface which provides maximum stability.
- 3) Item may be used indoors if desired.

DISPOSAL

- Please respect the environment!
- Dispose of the packaging according to the local recycling regulations.
- Dispose of paper and cardboard packaging separately from plastic bags into the corresponding containers.
- Dispose of the appliance at the end of its life span according to environmental considerations.

WARNING

Check the item as soon as it arrives. Do not use the item if it has been damaged during shipping. Always use extreme caution when walking up or down the step. Use extra caution if step is wet.

TECHNICAL SPECIFICATION

Maximum user weight: 440lbs (200kgs / 31.4 Stone). Measurements: L49xW39xD10cm.

Imported and distributed by Easylife Group Ltd, London NW1 0JP

Customer service: 0303 031 0777

For more innovative products from Easylife please visit:

www.easylifegroup.com



Half Step INSTRUCTION MANUAL

A small
step for
greater
mobility



Model No. - EL4598

Please read carefully before use and keep for future reference