

Super Mini Ear

INSTRUCTION MANUAL

Model No.5116

Please read carefully before use and keep for future reference

PLEASE READ THE INFORMATION ON THE FOLLOWING PAGES BEFORE USING YOUR HEARING DEVICE

The hearing device will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Infrequent use of a hearing device does not permit a user to attain full benefit from it.

The use of a hearing device is only part of hearing habitation and may need to be supplemented by auditory training and instruction in lipreading.

HAZARD WARNINGS


- Changes or modifications to the hearing device that were not explicitly approved by seller are not permitted. Such changes may damage your ear or the hearing device.
- Hearing device batteries are toxic if they are swallowed! Keep out of reach of children and pets. If batteries are swallowed, consult your medical physician immediately!
- If you feel pain in or behind your ear, if it is inflamed, if skin irritation or accelerated accumulations of ear wax occur, please check with your hearing care professional or physician.
- **BEWARE:** The hearing device contains small parts which can be swallowed by children.
- Do not take the hearing aid apart.
- Please consult your doctor for any health conditions you are experiencing before using this product.

INFORMATION ON PRODUCT SAFETY

- Never immerse your hearing device in water. Protect it from excessive moisture. Always remove your hearing device before showering, bathing, or swimming.
- Protect your hearing device from heat (never leave it near a window or in the car). Never use a microwave or other heating devices to dry your hearing device. Ask your hearing care professional about suitable drying methods.
- Store the hearing device in its case and in a safe, dry, and clean place.
- Do not drop your hearing device. Dropping onto a hard surface can damage it.
- Remove the battery if you are not using your hearing device for a long period of time.
- X-ray radiation (e. g. CT scans, MRI scans) may adversely affect the correct functioning of your hearing devices. We recommend that you remove them before undergoing X-ray procedures and keep them outside the room.
- Do not use your hearing device in areas where electronic equipment is prohibited.

YOUR DEVICE

- A) Ear Cap
- B) Super Mini Ear
- C) Volume Control
- D) Microphone
- E) Battery compartment

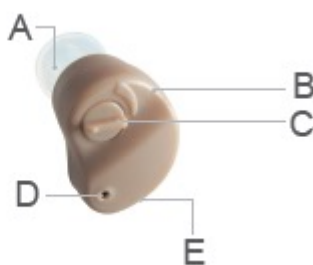

Model	JH-906
Output. OSPL ₉₀ Max.	≤133dB
FOG50 MAX Gain	≤45+5dB
EQ Input Noise	≤50dB
Frequency Range (Not narrower than)	200Hz-4600Hz
Working Current	≤3.5mA
Total Harmonic Distortion	≤30%
Rated Volatage	D.C 1.5V
Battery Size	AG3/LR41H

EasyLife Ltd, London, NW1 0JP

Support Line: 0333 030 0777

© EasyLife Ltd. 2022 (Reg. No. 05221840)

For more innovative products, please visit

www.easylife.co.uk

IN5116 ©2022 EasyLife Ltd

 **easylife**
everyday solutions