BIO POSTURE





- Relieves back, neck & shoulder pain
- Supports & improves posture
- Reduces muscular tension



World beating technology

The only spinal correction system **designed and endorsed** by **World Leading Biomechanics expert** Prof. Malcolm Pope Dr.Med.Sc., Ph.D., D.Sc., C.Eng., Eur Ing., Eur Erg.

BIO POSTURE

BACK CORRECTOR


WITH BIOFFEDBAC™TECHNOLOGY

Fast relief from back muscle pain

Biofeedbac[™] takes the latest health innovation techniques in sports science drug-free medicine and applies them to the home environment.

- Proven FAST pain relief
- Clinically tested
- Prevents muscle damage
- Improves & supports posture
- Reduces muscle tension
- Lightweight & comfortable
- DRUG FREE & no electronics
- SAFE with pacemakers
- Developed by a spinal expert
- Wear discreetly under clothing

INSTANT RELIEVE FROM POOR POSTURE PAIN

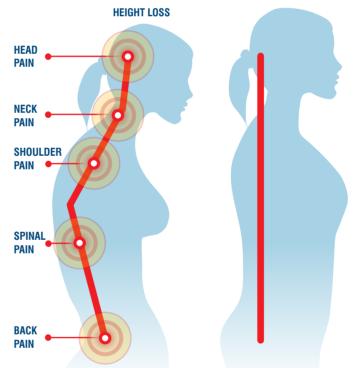

POOR PAINFUL POSTURE

BIOFEEDB AC

ELIMINATES BACK PAIN

STRAIGHT AWAY!

Comfortable shoulder pressure eases

and strengthens drooping shoulders, bringing them back into alignment.

Well-constructed and comfortable

Back panel adjustable neoprene

and velcro padding, keeps the

lower back aligned and strong

and can fit any waistline. An

extender belt is also available.

SHOULDERS •

QUALITY BUILD


under or over clothes

FIRM SUPPORT

HEALTHY POSTURE

BIOFEEDBAC

How to wear a Bioposture Back Corrector

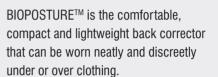

Easy to put on and take off with adjustable velcro straps. Fits all sizes from waists 22" to 55".

HOW TO FIT

Remove your upper clothing, or wear something light, so that your lower back and front is clear of bulky clothing.

You do not need to remove your lower body clothing.

- Put your arms through the arm straps and adjust each strap so that the waist band is in the correct position.
- 2 Place the Bioposture Back Corrector to your lower back, making sure that the side with green pads faces your skin. Position the belt so that the protective centre panel is placed against your lower spine.
- Close the belt around your waist. Ensure it is a snug fit, but not so tight as to be uncomfortable. The belt should be just above the belt line.

WARNING: Ensure the belt and shoulde straps are not too tight as this could cause bruising or soreness to skin.

POSITION: Correctly fitted belt and strap positions are shown in the image to the right.



















The neck is free to move and benefits from the strength-building technology which aligns the spine and muscles surrounding it.

COMFORTABLE FIT

Fully adjustable straps for all body types.

SPINE

Back panel comfortably eases the spine straight.

LOWER BACK

Firm, comfortable pressure is applied to the lower back, whilst Biofeedbac technology works with the body's own proprioceptive system to re-educate the muscles around the spine.



Made in China exclusively for: Bioenergiser Ltd. Worcestershire, DY11 7RA, UK www.bioposture.tv Rockin Direct, Melbourne, VIC 3207, AU www.biofeedbac.com.au

BIOFEEDBAC

C€ BE276