



## DISPOSAL

- Please respect the environment!
- Dispose of the packaging according to the local recycling regulations.
- Dispose of the appliance at the end of its life span according to environmental considerations.

## WARNING

This product is intended for domestic use only.

## TECHNICAL SPECIFICATION

Measurements: 55mm x 135mm High x 80mm Depth



# Battery Operated Ear Cleaner

## INSTRUCTION MANUAL



Model No. EL6181

Please read carefully before use and keep for future reference

Imported and distributed by EasyLife Group Ltd, London NW1 0JP

**Customer service: 0844 441 8448**

For more innovative products from EasyLife please visit:

[www.easylifegroup.com](http://www.easylifegroup.com)

IN6181-1 © 2015 EasyLife Group Ltd



**Customer service: 0844 441 8448**  
For more innovative products from EasyLife please visit:  
[www.easylifegroup.com](http://www.easylifegroup.com)



### Dear Customer

Thank you for purchasing the Battery Operated Ear Cleaner.


**Ear hygiene has taken a major step forward. Out go cotton buds - they can cause infection and push wax further into the ear - in comes this clever CE approved Ear Cleaner. Using gentle suction it clears out wax and dirt without poking and prodding and risking ear damage. The 4 silicone tips are soft and easily washable for regular use. The freestanding Ear Cleaner comes with a filter, special cleaning brush. Requires 2xAA batteries (not included).**

### INSTRUCTIONS

The Battery Operated Ear Cleaner is a safe and effective way of cleaning wax from your ears. A great advance in personal hygiene, no more poking around with cotton buds. Please read these instructions carefully before use and keep them in a safe place for future reference.

### CAUTION

- Do not use if you are suffering any ear pain or from an ear infection.
- Do not use for anything other than the intended use.
- Do not store near any heat source.
- Always store in a dry place.
- Do not immerse in water.
- Keep out of reach of children.

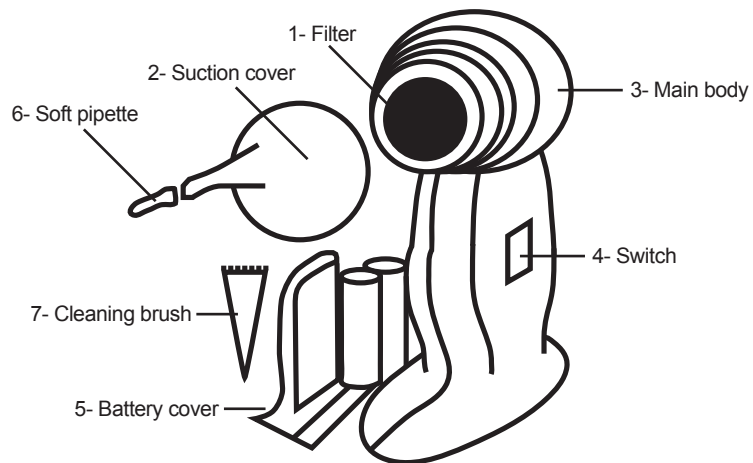

### PARTS


- |                  |                    |
|------------------|--------------------|
| 1- Filter        | 5- Battery cover   |
| 2- Suction cover | 6- Soft pipette x4 |
| 3- Main body     | 7- Cleaning brush  |
| 4- Switch        |                    |

### INSTRUCTIONS FOR USE

1. Insert 2x AA batteries. See diagram 1
2. Fit the soft pipette to the end of the Ear Cleaner, please ensure it is secure. See diagram 2.
3. Slowly and gently insert the soft pipette into the ear. See diagram 3.
4. Switch on and the Ear Cleaner will gently remove wax from your ear.



1.

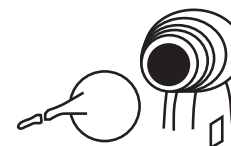

2.



3.

### AFTER USE

- Wash soft pipette and suction cover with warm soapy water.
- Rinse with clean warm water.
- Wipe dry with a clean cloth.

